

HUMAN ASSAYED MULTI-SERA - LEVEL 3 (HUM ASY CONTROL 3)

CAT. NO. HE1532	GTIN: 05055273203608	SIZE: 20 x 5ml
CAT. NO. HS2611	GTIN: 05055273203813	SIZE: 5 x 5ml
LOT NO. 1212UE	EXPIRY: 2025-08-28	

INTENDED USE

This product is intended for in vitro diagnostic use, in the quality control of diagnostic assays. The Human Assayed Multi-sera is for the control of accuracy.

DEVICE DESCRIPTION

The Human Assayed Multi-sera is supplied at 2 levels, level 2 and 3. Target values and ranges are supplied for the analytes listed in the values section at both levels.

SAFETY PRECAUTIONS AND WARNINGS

For in vitro diagnostic use only. Do not pipette by mouth. Exercise the normal precautions required for handling laboratory reagents.

Human source material, from which this product has been derived, has been tested at donor level for the Human Immunodeficiency Virus (HIV 1, HIV 2) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NONREACTIVE. FDA approved methods have been used to conduct these tests.

However, since no method can offer complete assurance as to the absence of infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and disposed of accordingly.

Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

OPENED: Store refrigerated (+2°C to +8°C). Reconstituted serum is stable for 8 hours at +15°C to +25°C or 7 days at +2°C to +8°C, and 28 days when frozen once at -18°C to -24°C. (See Limitations)

UNOPENED: Store refrigerated (+2°C to +8°C). Stable to expiration date printed on individual vials.

LIMITATIONS

For Total & Prostatic Acid Phosphatase, the material should be stabilised by adding 1 drop (25µl - 30µl) of 0.7M Acetic acid solution to 1ml of the serum exactly 30 minutes after reconstitution. After stabilisation Total and Prostatic Acid Phosphatase is stable for 2 hours at +15°C to +25°C, 2 days at +2°C to +8°C, and 28 days when frozen once at -18°C to -24°C.

Alkaline Phosphatase levels in the reconstituted serum will rise over the stability period. It is recommended that the reconstituted serum is allowed to stand for 1 hour at +15°C to +25°C before measurement.

Bilirubin in the serum is light sensitive and it is recommended that the serum is stored in the dark. Stored in the dark, it is stable for 4 days at +2°C to +8°C. Do not store at +15°C to +25°C. Do not freeze.

GLDH is stable for 2 days at 2-8°C.

NEFA is stable for 1 day at +2°C to +8°C.

Total PSA is stable for 4 days at +2°C to +8°C, or 28 days in aliquots frozen at -18°C to -24°C.

Bacterial contamination of the reconstituted serum will cause reductions in the stability of many components.

Different lot numbers of this control should not be interchanged, as the values assigned to the controls vary from lot to lot.

The control should not be used as a calibration material.

Due to the zinc content in some batches of rubber stoppers, the QC and calibrator material should be aliquoted into polypropylene tubes and stored at +2°C to +8°C to ensure stable zinc levels throughout the stability period.

PREPARATION FOR USE

The Human Assayed Multi-sera is supplied lyophilised.

1. Carefully reconstitute each vial of lyophilised serum with exactly 5ml of distilled water at +15°C to +25°C. Close the bottle and allow to stand for 30 minutes before use. Ensure contents are completely dissolved by swirling gently. Avoid formation of foam.

Do not shake.

2. Refer to the Control section of the individual analyser application.

3. Refrigerate any unused material. Prior to reuse, mix contents thoroughly.

MATERIALS PROVIDED

Human Assayed Multi-sera - Level 3 20 x 5ml / 5 x 5ml

MATERIALS REQUIRED BUT NOT PROVIDED

Volumetric pipette

ASSIGNED VALUES

Due to the variation caused by test equipment, test reagents and laboratory technique, the quoted ranges are provided for guidance. It is recommended that these ranges are used until each laboratory has established its own ranges, based on individual laboratory requirements.

Each batch of assayed human serum is submitted to reference laboratories for assignment against international Reference Standards.

Where international Reference Standards are unavailable, Reference Methods are used. Values are also collected from approx. 3000 laboratories worldwide and using a unique statistical analysis, a value is assigned.

With each batch, a control range is provided for individual parameters and each parameter method. The control range is equivalent to the assigned mean \pm 2S.D.

If an instrument specific value is not available, refer to the Method section. If necessary, contact Randox Laboratories - Technical Services, Northern Ireland, tel: +44 (0) 28 9445 1070 or email Technical.Services@randox.com.

NOTES

* All trademarks recognised.

(1) Applies only in Germany. Ranges established according to the Guidelines of the Federal Chamber of Physicians in Germany.

(2) Values established by reference laboratories officially recognised by the Federal Chamber of Physicians in Germany.

(3) DGKC: German Society for Clinical Chemistry.

(4) IFCC: International Federation of Clinical Chemistry.

(5) SCE: Scandinavian Committee on Enzymes.

METHOD
HUMAN ASSAYED MULTI-SERA - LEVEL 3

Lot. No.	1212UE	EXPIRY:	2025-08-28	Cat. No.	HE1532 / HS2611			
Analyte	unit	Target	low	high	1SD	2SD	methods	
Albumin	g/dl	3.05	2.59	3.51	0.23	0.46	Bromocresol Green	
Alkaline Phosphatase	U/L	546	464	628	41	82	Diethanolamine buffer DEA 37°C	
ALT (GPT)	U/L	127	102	152	12.5	25	Tris buffer without P5P 37°C	
AST (GOT)	U/L	136	109	163	13.5	27	Tris buffer without P5P 37°C	
Amylase Total	U/L	286	243	329	21.5	43	Siemens - blocked pNPG7 37°C	
Apolipoprotein A-1	mg/dl	93.4	76.6	110	8.4	16.8	Immunoturbidimetric	
Apolipoprotein B	mg/dl	72	59.0	85.0	6.5	13	Immunoturbidimetric	
Bilirubin Direct	mg/dl	1.83	1.44	2.22	0.20	0.39	Dichlorophenyl Diazonium (DPD)	
	mg/dl	1.93	1.53	2.33	0.20	0.40	Diazo with Dichloroaniline (DCA)	
Bilirubin Total	mg/dl	4.80	3.79	5.81	0.51	1.01	Dichlorophenyl Diazonium (DPD)	
	mg/dl	5.39	4.26	6.52	0.57	1.13	Diazo with Dichloroaniline (DCA)	
Calcium	mg/dl	12.7	11.4	14.0	0.65	1.3	Arsenazo III	
	mg/dl	12.7	11.5	13.9	0.6	1.2	Cresolphthalein complexone (CPC)	
Cholesterol	mg/dl	286	249	323	18.5	37	Cholesterol Oxidase	
Chloride	mmol/l	119	110	128	4.5	9	Colorimetric	
CK Total	U/L	503	413	593	45	90	CK-NAC (IFCC) 37°C	
Copper	µg/dl	169	135	203	17.0	34	Colorimetric	
Creatinine	mg/dl	4.12	3.30	4.94	0.41	0.82	Jaffe rate blanked	
	mg/dl	4.24	3.39	5.09	0.43	0.85	Enzymatic UV method	
gamma-GT	U/L	183	155	211	14.0	28.0	Gamma Glutamyl-3-Carboxy-4-nitroanilide (IFCC) 37°C	
Glucose	mg/dl	276	234	318	21	42	Glucose oxidase	
HDL - Cholesterol	mg/dl	101	85.3	117	7.85	15.7	Direct HDL PPD	
Immunoglobulin A	mg/dl	180	135	225	22.5	45	Immunoturbidimetric	
Immunoglobulin G	mg/dl	628	515	741	56.5	113	Immunoturbidimetric	
Immunoglobulin M	mg/dl	60.4	48.3	72.5	6.05	12.1	Immunoturbidimetric	
Iron	µg/dl	212	174	250	19	38	Colorimetric without ppt.	
Lactate	mg/dl	52.6	43.2	62.0	4.7	9.4	Colorimetric Lactate Oxidase	
LD (LDH)	U/L	790	671	909	59.5	119	P->L German methods 37°C	
Lipase	U/L	58	47	69	5.5	11	Other Colorimetric 37°C	
Lithium	mmol/l	1.97	1.73	2.21	0.12	0.24	Ion selective electrode	
	mg/dl	1.37	1.20	1.54	0.09	0.17		
Magnesium	mg/dl	4.25	3.74	4.76	0.26	0.51	Xylidyl Blue	
NEFA	mmol/l	0.51	0.41	0.61	0.05	0.10	Colorimetric	
Phosphorus	mg/dl	7.22	6.14	8.30	0.54	1.08	Phosphomolybdate UV	
Potassium	mmol/l	5.95	5.48	6.42	0.24	0.47	ISE method - direct	
	mmol/l	6.08	5.59	6.57	0.25	0.49	ISE method - indirect	
Sodium	mmol/l	154	147	161	3.5	7.0	ISE method - direct	
	mmol/l	156	148	164	4.0	8.0	ISE method - indirect	
Protein Total	g/dl	4.59	3.67	5.51	0.46	0.92	Biuret reaction end point	
TIBC	µg/dl	257	203	311	27.0	54	Direct Colorimetric	
Transferrin	mg/dl	166	133	199	16.5	33.0	Immunoturbidimetric	
Triglycerides	mg/dl	267	225	309	21.0	42.0	Lipase/GPO-PAP	
UIBC	µg/dl	50.3	41.3	59.3	4.5	9.0	TIBC - FE	
Uric Acid (Urate)	mg/dl	9.26	8.0	10.5	0.61	1.22	Uricase Peroxidase with ascorbate oxidase @ 546nm	
Urea	mg/dl	118	100.0	136	9.0	18.0	Urease kinetic	
	mg/dl	55.3	47.0	63.6	4.15	8.30	BUN	
Zinc	µg/dl	227	182	272	22.5	45	Colorimetric	